


# Essilor Natural®

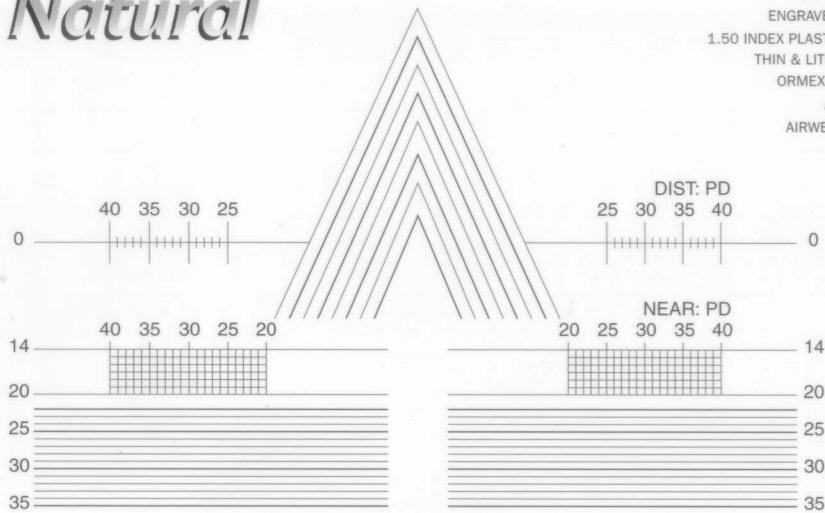

**TECHNICAL ASSISTANCE**  
**(800) 822-5367**

## LENS MARKINGS

ENGRAVED ADD: TEMPORAL SIDE  
 ENGRAVED CIRCLES: ○  
 ENGRAVED LOGO: NASAL SIDE  
 1.50 INDEX PLASTIC, GLASS, 1.50 TRANSITIONS®-Σ  
 THIN & LITE® 1.60 HIGH INDEX: Σ6  
 ORMEX® TRANSITIONS: Σ56  
 AIRWEAR®: ΣP  
 AIRWEAR® TRANSITIONS®  
 ORMEX: ΣX

## INSTRUCTIONS

### CENTERING:

- Mark/dot monocular PD & height on frame demo lenses
- Center frame on triangle with dots superimposed on "O" line
- Read R & L mono PD
- Read R & L mono height using lowest point of lower bevel

### SIZING:

Superimpose dot on fitting cross in lens diagram.  
 The frame must fall within the diameter of the circle for the material required.

### VERIFICATION:

Upon receipt of glasses from the lab, check markings using the centering instructions to ensure proper measurements. If lenses are not marked, they must be remarked before verification.



©2004 Essilor of America, Inc. Essilor, Airwear and Ormex are registered trademarks of Essilor International, S.A. PhotoGray Extra is a registered trademark of Corning Incorporated, Corning, N.Y. Transitions is a registered trademark of Transitions Optical, Inc. Essilor Natural is a registered trademark of Essilor International, S.A. Thin & Lite is a registered trademark of Essilor of America, Inc. 08/04CP

